



Surgical perfection. For life.

Register your product for the full warranty to take place

www.stille.se/iQ2registration

imagiQ2 installation procedure

Thank you for purchasing the STILLE imagiQ2 surgical imaging table. Below you find the steps that guide you for a successful installation and use of the product. For more info please contact STILLE at imagiQ2@stille.se

1. Unpack the table

For instructions on how to unpack the table please visit: www.stille.se/movies/imagiq2-product-training/ and select the video called "Unpacking imagiQ2" from the Product training video section.

2. Complete the installation report

Please be informed that the standard warranty terms will only apply if the table is registered with STILLE. In case the product is not registered with STILLE, the products are subject to a 1-year warranty from the date of delivery.

The installation report shall be submitted to STILLE no longer than 10 days from the date of the installation. For registering your product please visit: www.stille.se/iQ2registration

If you have any questions, please contact your sales representative or STILLE directly.

3. Read the user manual / Get to know the product


Read carefully and always follow the instructions in the manual before first using the imagiQ2™ operating table. This manual has been written to ensure that you will handle the table in a safe and correct manner. You will find the user manual included with the table. You can also contact us at imagiQ2@stille.se. STILLE has prepared an educational video channel for a quick and more hands-on product training. Each video is no longer than 1 minute and includes all table functionality, attachment of the accessories and packing/unpacking of the table. The educational video channel is not intended to replace the user manual.

Go to STILLE's educational video channel here: www.stille.se/movies/imagiq2-product-training/

4. FIRST TIME USE procedure

Follow the procedure described below before the STILLE imagiQ2™ operation table is used for the first time:

- 4.1) Charge the batteries continuously until they are fully charged or for a minimum of 12 hours for standard battery and 24 hours for battery upgrade before first time use (the main switch must be "ON" when charging the table).

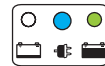

Constant Blue and Green light

- 4.2) Fill out the testprotocol ISO -EN 62353 (this protocol depends on each country requirements)
 - 4.1) Consult the user manual /educational video channel.
 - 4.2) Test run all functions on the hand control as described:
 - A. Column, Run column from lowest position to highest position and up again.
 - B. Lateral tilt right, Run tabletop from zero position to maximum right inclination, and then zero the table.
 - C. Lateral tilt left, Run tabletop from zero position to maximum left inclination, and then zero the table.
 - D. Longitudinal tilt (Trendelenburg), Run tabletop from zero position to maximum head down, and then zero the table.
 - E. Reversed longitudinal tilt (anti-Trendelenburg), Run tabletop from zero position to maximum head up and then zero the table.
 - F. Quick-trend (Trendelenburg), Run tabletop from zero position to maximum head down and then zero the table.
 - G. Zero position. Lateral tilt, Run tabletop from maximum left inclination to maximum right inclination and back. Make sure that the table stops both times it passes zero position.
 - H. Longitudinal tilt (Trendelenburg) Run tabletop from maximum head down to maximum head up and back. Make sure that the table stops both times it passes zero position.
 - I. Panning (float). Pan functions, place table in 0-position and test run both axis. Check that motion works effortlessly and smooth (manoeuvre with one hand only).

All above mentioned steps (A-I) shall be controlled on the safety Hand Control as well.



To learn more about the imagiQ2's functionality please consult the user manual or watch the educational video channel.

Daily control before procedures

- The operator should have knowledge how to operate the device, be aware of warnings and signals described in the user manual and the user manual should always be available. For a video presentation of the table use please consult the educational video channel at: www.stille.se/movies/imagiq2-product-training/
- The operator should perform a function according to steps A-I (See FIRST TIME USE procedure).
- Test the device for proper function and inspect cables for cuts and other damage. If in doubt, replace the relevant parts.

Good to know:

- The main switch should be in the "Up" position to start the table (it is shipped in the "off/down" position) To find the main switch on the base of the table please consult the "Main switch (power ON & OFF)" educational video at: www.stille.se/movies/imagiq2-product-training/
- The pan handle has a tension adjustment. Make sure you keep it tight to the side rail.
- The table can also be operated with AC power. STILLE recommends for the battery to be fully charged before first time use.
- Make sure the battery is properly charged during at least 12 hours for standard battery /24 hours for battery upgrades before first time use.

For more information please contact STILLE:

Customer Service USA
Phone: +1 (800) 665-1614
E-mail: sales.us@stille.se

Technical Service USA
Phone: +1 (800) 665-1614 ext. 105
E-mail: service.us@stille.se

Customer Service International
Phone: +46 (8) 588 58 011
E-mail: imagiQ2@stille.se

Technical Service International
Phone: +46 (8) 588 58 030
E-mail: service@stille.se